

## SLIM SUPPORTS

Customizes a standard wheelchair or geri-chair for petite patients/residents. The Slim Supports fit snugly inside the seating area to provide bilateral torso support. Made of 3/8" (1 cm) plywood covered with padded foam and wipe-clean vinyl. Adjustable straps for attachment. One pair per package. Meets California #117 Flame Retardancy Standards



- REF 6322** Wheelchair 16"D x 5"H (front) - 12"H (rear) (41 cm x 13 cm - 31 cm)
- REF 6323** Geri-chair 23"D x 7"H (front) - 12"H (rear) (58 cm x 18 cm - 31 cm)

## MONITORING

**AWARNING** Always follow your facility's policies and procedures for patient assessment, monitoring and rehabilitation

## INSPECTION

**AWARNING** Inspect before each use: check for broken stitches or parts; torn, cut or frayed material; or zippers or buckles that do not hold securely. DO NOT use soiled or damaged products.

## WHEELCHAIR APPLICATION INSTRUCTIONS:

1. Slim Supports are provided in pairs (right and left) and should be placed between the wheelchair side panel and the patient. Place the zippered edge facing down and the hard side of the support toward the side panel.
2. To secure the Slim Support, work the bottom front strap between the side panel and the seat rail until it clears the bottom of the arm. Pull the lower strap tightly, then connect it to the top strap on the outside of the side panel. To secure the straps, loop the bottom strap through the rings on the top strap (fig. 1) and pull tightly.
3. Wrap the rear strap around the rear post and secure it using the hook and loop closure.
4. Repeat the process for the opposite Slim Support.

## GERI-CHAIR APPLICATION INSTRUCTIONS:

1. Slim Supports are provided in pairs (right and left) and should be placed between the side panel and the patient.
2. In fig. 2, Slim Supports are shown in the horizontal position. This is more appropriate for shorter patients.
3. In fig. 3, Slim Supports are shown in the vertical position. This is more appropriate for taller patients.

## CLEANING INSTRUCTIONS (ALL):

- Wipe Clean with mild detergent. OSHA approved intermediate level disinfectants can be used per manufacturer instructions. DO NOT use phenol and benzyl based disinfectants.
- After cleaning, products MUST be rinsed with water to remove any residual chemicals.
- Make sure products are completely dry before use.



## DISPOSAL

**AWARNING** Properly dispose of the product per facility's policy for BIOHAZARDOUS materials.



## STORAGE AND HANDLING

This device is designed for use in normal indoor environments. This device may be stored in ambient warehouse temperatures at normal humidity levels. Avoid excess moisture or high humidity that may damage product materials.