

## WHEELCHAIR WING-BACKS

These side lateral supports help provide a vertical side support surface for patients who lean laterally. The angle of attachment may be adjusted manually to accommodate the needs of the individual patient/resident. Made of 1/4" (1 cm) plywood base covered with foam and a vinyl cover. The Posey Wheelchair Wing-Backs install in seconds using the existing screws from the wheelchair back upholstery.



- Meets CA #117 flame retardancy standards.
- Mounting hole spacing, 5 1/4" (13 cm)
- One pair per package.

**REF 6321** 8"W x 10"L (20 cm x 25 cm)

## MONITORING

**▲WARNING** Always follow your facility's policies and procedures for patient assessment, monitoring and rehabilitation

## INSPECTION

**▲WARNING** Inspect before each use: check for broken stitches; or torn, cut or frayed material. DO NOT use soiled or damaged products.

## APPLICATION INSTRUCTIONS:

1. Remove the top two screws on each side of the wheelchair backrest.
2. Orient the Wing-back so that the screw holes in the metal brackets line up with the screw holes of the wheelchair back post. The Wing-back should be positioned so that it is above the wheelchair armrest.
3. Attach the screws so that the seat back and the Wing-back are both secure.

The angle of the Wing-back may be adjusted by applying pressure to the metal bracket and bending it slightly. To avoid damaging the Wing-back, be sure to brace the whole product with your hand before applying pressure to the metal bracket.

## DISPOSAL

**▲WARNING** Properly dispose of the product per facility's policy for BIOHAZARDOUS materials.

## STORAGE AND HANDLING

This device is designed for use in normal indoor environments. This device may be stored in ambient warehouse temperatures at normal humidity levels. Avoid excess moisture or high humidity that may damage product materials.

## CLEANING INSTRUCTIONS:

- Wipe Clean with mild detergent. OSHA approved intermediate level disinfectants can be used per manufacturer instructions. DO NOT use phenol and benzyl based disinfectants.
- After cleaning, products MUST be rinsed with water to remove any residual chemicals.
- Make sure products are completely dry before use.

